beurer

HK 25, 35, 63 110 V

HK 25

HK 35

HK 63

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ENGLISH

Dear Customer,

We are happy that you have decided on a product from our range. Our name stands for high-quality and thoroughly tested products from the areas of warmth, blood pressure, body temperature, weight, gentle therapy, massage and air.

Please read these instructions for use carefully and follow the directions.

Best regards,
The Beurer Team

Items included in the package: 1 heating pad, 1 switch, 1 cover, these operating instructions

1. Important safety information – read carefully and keep for later use

Failure to observe the information provided below can lead to personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information not only serves to protect your health and the health of others, it also protects the product from damage. Therefore, observe this safety information and pass on these instructions together with the product.

- Only use this heating pad for the purpose described in these instructions.
- This heating pad is not intended for use in hospitals.
- This heating pad is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety.
- This heating pad should not be used with persons who are helpless, small children or persons insensitive to heat (e.g. diabetics, persons with skin changes related to illness or persons with scarred skin in the area where the pad is to be applied, after taking any pain-relieving medication or alcohol).
- Excessively long use can cause the skin to be burned.

- Caution! Never fall asleep while the heating pad is switched on.
- Do not apply the heating pad to parts of the body which are inflamed, injured or swollen. If in doubt, seek medical advice before use.
- The electric and magnetic fields emitted by this heating pad can interfere with the operation of pace-makers. However, they are far below the permissible limits: electric field strength: 5,000 V/m max., magnetic field strength: 80 A/m max., magnetic flux density: 0.1 millitesla max. Therefore, please consult your doctor and the manufacturer of your pacemaker before using this heating pad.
- Heating pad
 - only connect to the voltage specified on the appliance,
 - only use in conjunction with switch types specified on the heating pad,
 - never operate without supervision,
 - never switch on when folded or compressed,
 - never wedge between objects,
 - never make any sharp creases,
 - never use with animals,
 - never use when damp,
 - the switch and mains cable should not be exposed to moisture.
- When in use, the electronic components in the switch of the heating pad make the switch slightly warm. The switch should therefore never be covered or placed on the heating pad when it is being operated.
- Do not pull, twist or make any sharp kinks in the cables.
- Do not stick pins or pointed objects into this heating pad.
- This appliance may not be used by children unless the switch has been preset by a parent or a supervisory person, or the child has been sufficiently instructed in how to use this heating pad safely.
- Children should be supervised to ensure that they do not use the heating pad as a toy.
- Please check this heating pad frequently for signs of wear and tear or damage. If there are signs indicating the above or if this heating pad was used improperly, it must be returned to the manufacturer or retailer before further use.
- If the mains connection cable of this heating pad is damaged, it must be replaced by the manufacturer, its customer service department or a similarly qualified person to avoid any risk.
- Repairs may only be carried out by specially trained personnel or at an authorised repair workshop, as special tools are required. Incorrect repairs may cause considerable danger to the user.
- Should you still have questions on using our appliances, please contact our customer service.

2. Proper Use

This heating pad is designed for warming the human body. It allows you to apply targeted warmth. Heat promotes the circulation and relaxes the muscles. This heating pad is not intended for use in hospitals or for commercial applications. In particular, it may not be used on infants, small children, persons insensitive to heat, helpless persons or animals.

To avoid soiling or damaging the heating pad, we recommend using a fabric cover, particularly when applying essential oils and ointments.

When used for the first time, the heating pad may start to smell of plastic but this will soon go away.

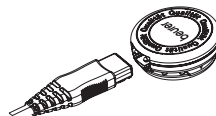
3. Operation

3.1 Safety

This heating pad has a SAFETY SYSTEM. This electronic sensor technology prevents the entire surface of the heating pad from overheating by automatically switching off in the event of a fault. If the safety system switches off the heating pad automatically as a result of a fault, the temperature display on the switch is no longer illuminated even when the heating pad is switched on. Please note that, after a fault has occurred, the back and neck heating pad can no longer be used for safety reasons and must be sent to the service address indicated.

3.2 Putting into operation

First check that the switch is connected to the heating pad at the plug-in connection. Plug the power cord into the mains outlet.

3.3 Selecting temperature

The fastest way to heat up the heating pad is to first set the switch to the highest setting. A lower setting can be selected later if required.

3.4 Temperature settings

- Level:
- 0 Off
 - 1 Minimum heat
 - 2 Medium heat
 - 3 Maximum heat

3.5 Automatic switch-off function

The heating pad switches off automatically after approx. 90 minutes. The illuminated temperature setting display then starts to flash. To switch on the heating pad again, first set to "0" and then after approximately 5 seconds select the desired temperature setting. Switch off the heating pad (0) or pull the plug out of the mains socket if you are not going to use the heating pad after the automatic switch-off.

3.6 Fast heating HK 35, 63

These heating pads have a fast heating function which allows them to heat up quickly within the first 10 minutes.

3.7 Additional information for HK 63

The heating pad is fitted with 6 integrated permanent magnets that produce a homogenous magnetic field for magnetic field applications.

4. Cleaning and Care

- **Before cleaning the heating pad, always unplug it from the mains.**
- The textile cover can be cleaned according to the symbols on the label, and for this purpose must first be removed. Also clean the plastic casing of the heating pad from time to time with a damp cloth.
- Please note that the actual heating pad may not be machine-washed, spun or machine dried, chemically cleaned, wrung out, pressed or ironed.
- Smaller stains can be removed using a cloth or moist sponge together with a little liquid detergent for delicate fabrics if necessary.
- **Only use the heating pad again once the heating pad and cover are completely dry.**

5. Storage

If you are not going to use the heating pad for some time, we recommend storing it in its original packaging in a dry place without weighing it down. To avoid breaking the heating pad, do not place any objects on top of it whilst storing.

Allow the heating pad to cool down beforehand.

6. Disposal

Please dispose of the blanket in accordance with the directive 2002/96/EG – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

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